



MEMORANDUM

To : CDL Clients
From : The Laboratory
Subject : Reminder: Preservative for 24h urine
Date : August 20th, 2012

Dear client,

CDL Laboratories, dedicated to producing high quality results, would like to remind its clients of the various 24 urine tests that require a preservative, in the 24 hr urine container, prior to specimen collection.

| CDL Code | Test Name | Preservative to include | Preservative volume |
|--------------------|---|---------------------------------|---------------------|
| HXPU | 24 hr Urine Hydroxyproline | HCl 6N | 25 mL |
| OXAL | 24 hr Urine Oxalate | HCl 6N | 25 mL |
| SO4/U | 24 hr Urine Sulfate | HCl 6N | 10 mL |
| UPORF (profile) | 24 hr Urine Porphyrine *Protect from light | Na ₂ CO ₃ | 5g |
| UA/U | 24 hr Urine Uric Acid | NaOH 1N | 10 mL |

For all other 24 hr urine testing, a urine container (for 24 hr collection) without preservative may be used.

It is essential to request the appropriate container when placing your supply order. The type of preservative and quantity may vary, depending on the test that is being ordered.

For all technical questions, please do not hesitate to contact the laboratory at **(514) 344-8022 ext. 223**.

The Laboratory.