

SPUTUM

Document number	LA-75-WI-110E_02.20
Revision date	2018-Jul-04
Required Specimen Type	<ul style="list-style-type: none"> • Expectoration sample./sputum
Medical Supplies	<ul style="list-style-type: none"> • Sterile container; • Biohazard bag;
General Guidelines	<ol style="list-style-type: none"> 1. All specimens must include a double identification (first and last name of the patient and date of birth or medicare number). 2. The use of gloves is required when handling all biological specimens. 3. A valid requisition must include the following information : <ul style="list-style-type: none"> • First and last name of the patient, date of birth or medicare number and gender. • Date and time of specimen collection and signature of collector. • Physician's full name, signature and license number (or other qualified prescriber). • Any relevant clinical information. • Requested tests.
Patient Preparation	<ol style="list-style-type: none"> 1. Patient should rinse mouth with water; 2. Any sputum with saliva or post-nasal secretions are not acceptable for testing; 3. Only specimens obtained from a deep cough and with thick consistency are acceptable for testing.
Specimen Collection Instructions	<ol style="list-style-type: none"> 1) Wear a protective mask; 2) Open the sterile container; 3) Cough to produce a deep expectoration by tapping several times on the chest; 4) Collect expectoration directly into the sterile container. A volume of at least 1mL is necessary; 5) Close the container tightly; 6) Identify the container with the following information: <ul style="list-style-type: none"> • First and last name; • Date of birth and Medicare number; • Date and time of collection. 7) Place the container in the biohazard bag; 8) Wash hands with soap and water; 9) Return the container <u>as soon as possible</u> to laboratory or physician's office.
Additional Information	N/A
Specimen Conservation and Stability	<ul style="list-style-type: none"> • 2 hours at room temperature • 20 to 24 hours refrigerated